

## WARRANTY SHEET

Final inspection:

Name:

Signature:

**The product is under warranty for period of  
24 months after date of purchase.**

**The warranty covers faulty material and  
processing defects.**

### Warranty conditions:

1. The product can be used entirely for the purpose it is designed and the instructions in the user guide must be followed.
2. The product has to be properly used, cleaned and stored in accordance to the user guide instructions.
3. The warranty does not cover: failures caused by overloading of the products, forced damage, unauthorized changes made on the device and/or its accessories and natural wear and tear of parts, especially upholstery dirt and damage, wear of plastic parts, brakes, tyre profile and tyre puncture.
4. The warranty claim has to be applied by the seller or authorised service agent of the device.
5. The properly filled warranty sheet and purchase confirmation must be declared by the warranty application.
6. The claim procedure can be started when the device is delivered complete and in origin packing, carefully packed and secured against damage. The owner is committed to deliver the device cleaned, washed and hygienic unobjectionable according general hygienic requirements. The basic maintenance and washing cannot be expected or even requested in the claim.
7. The warranty period does not stand for usage period, that means for the period when by correct use and maintenance the device in accordance to its features, purpose and construction could be used.
8. Neither producer nor its agent will be liable for any loses and injuries caused by manipulation and usage of the product in contradiction of the "User Guide" instructions.
9. The warranty is terminated whenever the following conditions arise: the user fails to follow the instruction mentioned in this user guide and manufacturer's storage and maintenance suggested procedures, or whenever the product is serviced by any other party except by the authorised service centres, or when the identification label is not stucked on the stroller and also in the warranty sheet.

Purchase date (day, month, year):

Name (type), phone number, stamp, signature of seller:

\_\_\_\_.\_\_\_\_.200\_\_

# patron®

## USER GUIDE

### OPTIONAL ACCESSORIES - REHAB STROLLERS (except of rehab-buggies all types)




#### Manufacturing and Distribution:

PATRON Bohemia a.s.  
Ceskolipská 3419  
CZ- 27601 Mělník, Česká Republika  
tel.: +420-315 630 111  
fax.: +420-315 630 199  
Info.cz@patro.eu  
[http:// www.patron.eu](http://www.patron.eu)

**Hood**

fit - remove the side paddings in the upper part of backrest and the hood brackets put into both hood attachments on the backrest frame, side parts of the hood upholstery fasten with plastic clips to the chassis, or unzip and put into the net pocket. The hood can adjust by pushing onto the hood frame.

remove - press the hood fittings releases and pull the hood out of the hood attachments could be rolled out.

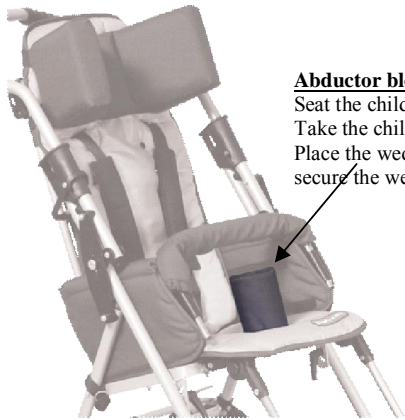

**Abductor block**

Seat the child into the stroller and find the appropriate place of the wedge. Take the child out of the stroller. Find the relevant fitting hole on the seat base. Place the wedge in the seat (the narrower part should aim to the backrest) and secure the wedge by a nut.

**Headrest (pair)**

Seat the child into the stroller and find an appropriate height of the headrest. Take the child from the stroller. Adjust the backrest height. Find the relevant fitting holes on the rear side of the backrest base. Put the screws of the rest through the holes in the upholstery and backrest base. Secure the headrest by nuts on the backrest.



**Headwings**

Sit the child into the stroller, to determine right height of headwings. Take the child out of the stroller. Put headwings in determined height onto the backrest, and secure it using straps with Velcro.

	TOM4	BEN4	TOM5
Hood	X	X	X
Abductor block	X	X	X
Headrest (pair)	X	X	X
Headrest adjustable COMFORT	N	N	X
Headwings with side fixation	X	N	X
Headwings BASIC	N	X	N
Headrest (pair) with neckrest	X	X	X
Side trunkrest (pair)	X	X	X
Hip rest (pair)	X	N	X
Lumbar support (width 25cm, 35cm)	X	X	X
Two points imm. belt	X	X	X
Four points imm. belt	X	X	X
Chest-belt addition to 4p-belt	X	X	X
Crotch-belt addition	X	X	N
Five point fixation belt	X	X	X
Chest imm. belt	X	N	N
Feet fixation	X	X	X
H - style harness	X	X	X
Imm. waistcoat	X	N	X
Abduction belt	X	X	X
Padded frame cover	X	X	N
FlexiPads	X	N	N
Rain cover	X	X	N
Legsmuff - summer	X	X	X
Legsmuff - winter	X	X	X
Workboard	X	N	X
Room base	X	N	X
Bag	X	X	X
Basket - fabric	X	X	X
Mini footrest	X	N	N
Extended side protection	X	N	N
Knee block	N	N	X
Pelvis fixation	N	N	X
Calf rest with fixation	N	N	X
Basket HD (steel)	N	N	X
Tilt lock for spastic children	N	N	X
Grab bar brackets cover	N	N	X
Hood - summer	N	N	X



**Basket - fabric**

Clip the basket on the chassis frame tubes. **Maximum load capacity is 3 kg.**

**Knee block**

Loosen two hexagon-bolts placed on the rear side of footrest slider profile and take off the footplate. Put on the knee block, set the position and fix the bolts into the assembly holes. Put on the footplate back into the right position and properly tighten both hexagon-bolts.

**Pelvis fixation**

Unfasten the front narrower belts from the upholstery and put them through the aperture in the side plate and upholstery of the seat. Attach the metal anchor by the nut which is fixing the position of the side plate. Unfasten the back wider belts from the upholstery and put them through the aperture between the side plate and seat base. Attach the metal anchor by the nut which is fixing the position of the side plate. Then fix the belts back to the upholstery and tighten them according to the need.

**Calf rest with fixation**

Loosen two hexagon-bolts placed on the rear side of footrest slider profile and take off the footplate. Put on the calf rest, set the position and fix the bolts into the assembly holes. Put on the footplate back into the right position and properly tighten both hexagon-bolts.

**Basket-HD (steel)**

Screw the attachment adaptors for basket-HD to the back axle. Attach the basket to these adaptors by screws with plastic rosace. Then attach the basket by belts with Velcro in front part of the stroller.

**Tilt lock for spastic children**

Unscrew backrest recline adjustment lever. Put tilt lock into the groove where the lever is moving and put the second part of tilt lock onto the screw of the lever. Then screw on the lever back.

**Grab bar brackets cover**

Take off the grab bar and put on the plastic plug onto the grab bar bracket. Then put on the elastic cover and fasten the rubber band onto the plastic nut placed on the bottom of the seat. Repeat the same procedure onto the other side.

**MAINTENANCE OF THE ACCESSORIES**


- if the accessories are damaged or in need of any repair, only the approved service centre is allowed for this service. Alternatively, PATRON Bohemia a.s. customer department is available for products information.
- only PATRON Bohemia a.s. replacement parts should be used on these accessories.
- make sure you are familiar with the operation of the accessories.
- all accessories can be washed by hands; alternatively the headrest fabric, siderests fabric, hiprests fabric, abductor block fabric, fixation waistcoat, H - style harness, abductor belt, chest fixation belt can be washed in washing machine (max. 40°)
- never clean with ammonia based, bleach based or spirit type cleaners
- after exposure to damp conditions leave in warm environment
- store the accessories in a dry and safe place.

In case of any repair on product needed or advise about particular functions, please contact below stated approved repair centre.

The repair centre is responsible as well for waste disposal of the device, accessory and packing.

**Headrest (pair) with neckrest**

Before fitting the headrest with neckrest adaptor study the instructions from previous chapter. For fitting the neckrest adaptor in an appropriate height use the dry zipper. Regarding strollers Tom 4 Xcountry and Tom 5 Streeter , open the head cushion first , attach the neckrest using Velcro and then close the cushion.

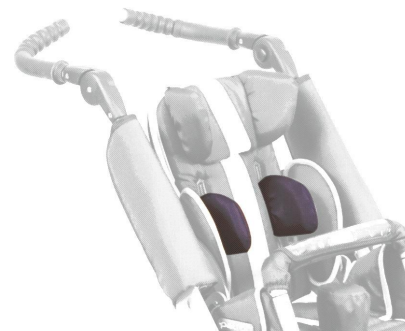


**Side trunkrest (pair)**

Seat the child into the stroller and find an appropriate height of the trunkrest. (The trunkrests should be in the occupant's arm pits.) Take the child from the stroller. Find the relevant fitting holes on the rear side of the backrest base and ( Regarding strollers Tom 4 Country and Tom 5 Streeter open zippers ) according to requested location cut the holes as big as metal plates of the side trunkrest through the fabric cover or artificial leather. Put the metal plates of trunkrests through the holes you cut in the upholstery. Put the screws through the holes in the backrest base and secure them by nuts. You can adjust the desired width of side trunkrests by the help of horizontal slots in the metal plates of side trunkrests.

**Hip rest (pair)**

Seat the child into the stroller and find the appropriate height of the rests. Take the child out of the stroller. Stick the dry zipper under the backrest upholstery in the relevant height. Attach the rests with the dry zipper to the stroller.

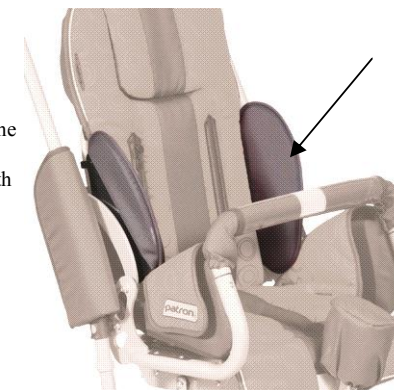

**Lumbar support (w.25cm, 35cm)**

Cut through leatherine vertical straps or fabric ( Regarding Tom 4 Xcountry and Tom 5 Streeter open zippers first ) in backrest in previously determined height, and 5 cm in width. Run Velcro through holes in backrest (w.5cm, 1.30cm), which is sewed on the lumbar support. Lift the backrest upholstery on the sides, and glue second part of Velcro on the metal plate - 2pcs.

### **Wedge of backrest**

Peel off the cover of the backrest (with Velcro). Stick the wedge of backrest to the desired height with Velcro. Stick the cover back.



### **Two points imm. belt**

**fit** - fix the belt in the lowest attachments at the rear of backrest (it is necessary to turn the belt to half width)  
**adjust** - unbrace the fastener, seat the child into the stroller and adjust the length of the belt. To adjust the belts length use the attachments at the rear of backrest. The occupant must always feel comfortable when using the belt.

### **Four point immobilisation belt**

**Mounting** - run the lower straps between backrest and side cushions and secure it using cam buckles at the rear side of the backrest. (it is necessary to turn the belt to half width).

Run upper straps through appropriate in backrest and fix them using upper cam buckles at the rear of backrest. In case that there would be two accessories fixed in the cam buckles, run straps through cam buckle-sonly once, otherwise run them through the buckles backwards to fix them.

**Setting and manipulation** - release all three fasteners of the belt, seat the child into the stroller and fix the clamps. Set desired straps length using plastic attachments, so that the belt would fixate, but not throttle the child. Other manipulation with the belt is to be made using plastic clamps.




detail



### **Chest-belt addition to 4p-belt**

The chest-belt addition put on the upper (thinner) straps of the four point belt.

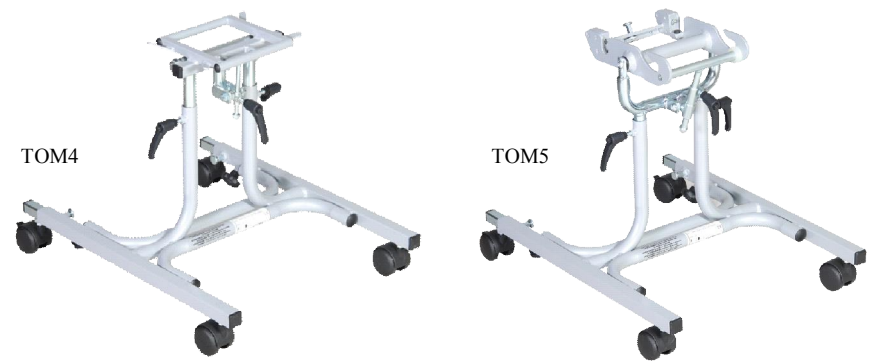

### **Room base**

Before fitting the seat to the room base study the user guide of your stroller, chapter "Fitting the seat in the ride direction".

**Distance between front (1) and rear - adjustable (2) wheels of the base must be sufficient for avoiding tipping the room base while using with the seat. The footrest of the stroller must be above the front wheels of the base.**

### **Extended side protection**

**Mounting** - the extended side protection slip-over the backrest and put it on the tubes of the handle from both sides with the aid of four plastic loops. Upper pair of loops put on between plastic girdle on the handle and the handle knob. The lapels of the extended side protection secure with velcro lock. Lower part of the extended side protection secure with strings.

**Seating position** - in seating position it is important that extended side protection is crossed from behind behind the backrest and secured with strap with velcro lock, this strap run in by plastic triangel near the upper loop and secure with velcro lock..

**Half-seating position** - extended side protection secure with the strap with velcro-lock that is run in by plastic triangel and buttoned up with velcro-lock in the back part.

**Lying position** - leave the extended side protection freely. In case you also buy the item hood, side hood upholstery button up the way that the hood upholstery would be between sides of the stroller and extended side protection.







**Legsmuff - summer**

Slip the muff over the footrest and attach it by the dry zipper bands to the handle tube.



**Legsmuff - winter**

Attach the muff by a rubber over the backrest (take off the hood when needed) and lash its middle part to the undercarriage. The band from artificial leather sewed in the muff allows use of the abductor block, 4-point imm. belt, 2-point imm. belt, abductor belt and H-style harness..



**Workboard**

Take off the safety bar. Place the work board attachments into the holes instead of the safety bar. Depth adjustment - release the screws underneath the board, adjust the depth and secure the screws properly.



**Bag**

Attach the bag to the handle by its studs. **Maximum load capacity 1 kg. Bags and rucksacks attached to the push handle affect the stability of the stroller.**

**Five point fixation belt**

**Mounting**

Run lower straps between backrest and side cushions and run them through lower cam buckles and secure them. In case that there would be two accessories fixed in the cam buckles, run straps through cam buckles only once.

Run upper (thin) straps through made holes and holes in the backrest panel, and run them through upper cam buckles, and secure them. In case that there would be two accessories fixed in the cam buckles, run straps through cam buckles only once (reversing - adjust according to your needs).

**Setting and manipulation**



Unlock the cam buckles and sit the child into the stroller. Determine where the crotch belt will be. Set length of straps using cam buckles, so that the belt would fixate, but not throttle the child. Take the child out of the stroller. Find appropriate holding hole on the seat metal plate for crotch belt placement, and cut through leatherine strip on seat accordingly. Run through this hole the hook with strip (without screw and nut). Put the screw onto the hook and hole in the metal plate, and secure it with nut on the bottom of the seat metal plate (in case you have abductor block, you do not have to run hook with strip through the upholstery, but you can fix it under the abductor block).

**Chest imm. belt**

**fit** - seat the child into the stroller and find the appropriate height of the belt. Take the child from the stroller. Fit the belt on backrest by belts, fasteners and velcro.

**usage** - undo the front velcro, seat the child into the stroller and fasten the velcro in order the belt positions the child. The occupant must always feel comfortable when using the belt.



**Feet fixation**

The belt is attached to the footrest with the dry zipper straps. The occupant's feet are immobilized by the dry zipper. The occupant must always feel comfortable by using the belt.

### **H-style harness**

**Mounting** - Run lower straps between backrest and side cushions and run them through lower cam buckles placed on the back plate of the backrest, in case that there would be two accessories fixed in the cam buckles, run straps through cam buckles only once, otherwise run them through the buckles backwards to fix them.

Run upper straps through these holes and run them through upper cam buckles placed on the back plate of the backrest. In case that there would be two accessories fixed in the cam buckles, run straps through cam buckles only once, otherwise run them through the buckles backwards to fix them.

**Setting and manipulation** - Open the buckles, fold up the fixation. Sit the child into the stroller, put the fixation on and secure the child using buckles. Adjust the length of the straps with plastic clip, so that the abduction belt would fixate, but not throttle the child. Other manipulation is to be made using buckles.

**Setting and manipulation** - Open the clips, fold up the fixation. Sit the child into the stroller, put the fixation on and secure the child using clips. Adjust the length of the strips with plastic clip, so that the abduction belt would fixate, but not throttle the child. Other manipulation is to be made using clips.



detail

### **Immobilisation waistcoat**

**Mounting** - Run side straps between backrest and side cushions. Run lower side straps through lower cam buckles which are positioned on the rear plate of the backrest, in case that there would be two accessories fixed in the cam buckles, run straps through cam buckles only once, otherwise run them through the buckles backwards to fix them.

Remove, remove slider buckles from upper straps and run straps through holes in upholstery and backrest plate. Put the buckles back on.

**Setting and manipulation** - Unzip the waistcoat, unlock buckles on higher straps. Sit the child into the stroller, zip the waistcoat and lock the buckles. Adjust the length of the straps with plastic clip, so that the abduction belt would fixate, but not throttle the child. Other manipulation is to be made using buckles

**Setting and manipulation** - Unzip the waistcoat, unlock clips on higher strips. Sit the child into the stroller, zip the waistcoat and lock the clips. Adjust the length of the strips with plastic clip, so that the abduction belt would fixate, but not throttle the child. Other manipulation is to be made using clips.




detail

### **Abduction belt**

**Mounting** - Run the straps with velcro through these holes (TOM: run straps with velcro through the gap between the backrest upholstery and seat). Lace the straps behind the back tube of the seat and secure using velcro.

Run upper thinner straps between backrest and side cushions and secure with upper cam buckles placed on the back plate of the backrest. In case that there would be two accessories fixed in the cam buckles, run straps through cam buckles only once, otherwise run them through the buckles backwards to fix them.

**Setting and manipulation** - Open the buckles and sit the baby into the stroller. Fasten the buckles. Adjust the length of the straps with plastic clip, so that the abduction belt would fixate, but not throttle the child. Other manipulation is to be made only using plastic buckles.

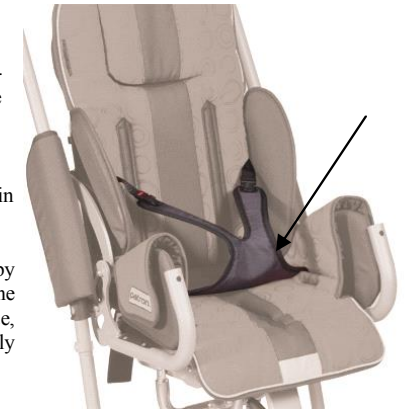

### **Padded frame cover**

Attach the cover over the handle tube and over the handle folding release with the dry zipper.



### **FlexiPads**

The set of six different cushions enable the seat width adjustments according to the occupant's needs.



### **Rain cover**

The rain cover should be pulled over the stroller, so the hood and also the footrest would be covered. Store clean and dry.